



JDENTAL CARE

*just smile*

# ***JDPTERYGO DRILLS KIT***

*less stress than ever on bones*




# JDPterygo Drills Kit


The JDPterygo Drills Kit contains drills specifically designed for the insertion of JDPterygo and JDPterygo One implants into the pterygoid bone. The kit contains longer drills for proper bone preparation in the apical and coronal areas. These drills allow a precise, safe and fast osteotomy.

## Drilling Protocols

Ø 3,3

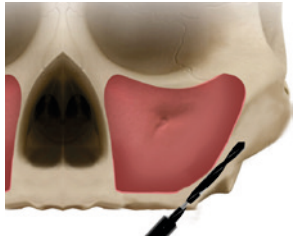

1. Start the osteotomy using JDPterygo drill Ø 2.0mm at the same implant length to be inserted

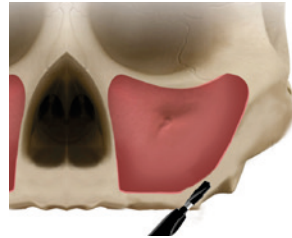

2. Complete the osteotomy with JDPterygo drill Ø 2.8mm at the entrance for 6mm


Ø 4,0



1. Start the osteotomy using JDPterygo drill Ø 2.0mm at the same implant length to be inserted.



2. Continue using JDPterygo drill Ø 2.4mm at the same implant length to be inserted.

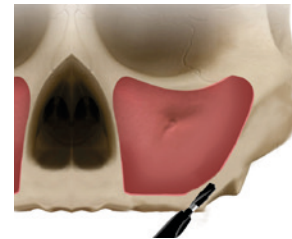

3. Complete the osteotomy with JDPterygo drill Ø 3.2mm at the entrance for 6mm.

## Drills for non guided protocol:

---

JDDRPT20	Drill Ø 2.0 JDPterygo
JDDRPT24	Drill Ø 2.4 JDPterygo
JDDR101	Drill Ø 2.8 JDPterygo
JDDRPT32	Drill Ø 3.2 JDPterygo



## JDPterygo Drills Kit:

---

JDKIT01	JD Surgical Kit - JDPterygo Drills Kit
---------	--



